

MLR50NCAL Universal Pull Down Grab Rail with Support Call Facility

Note


This product is to be installed by suitably qualified persons, in a fit-for-purpose application, to suitable materials, using suitable fixings, and complying with any relevant codes.

Dimensions

315mmH x 104mmW X 857mmD

Hardware Provided


Please be aware that fixings are not included. *It is the responsibility of the installer to ensure that the surface/substrate that the unit is installed upon and the fixings used are suitable to withstand the weight-bearing loads of the unit and if required as per AS 1428.1 / Australian building code regulations.*

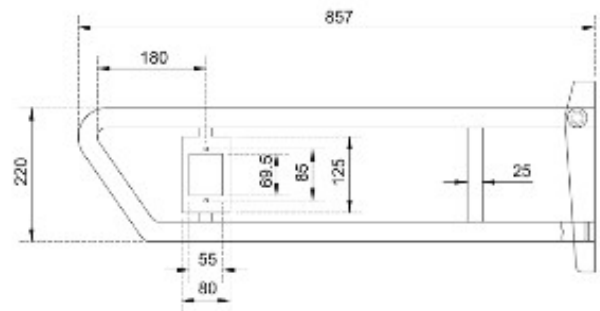

*Images above depict model MLR50NCAL_UNI_SS

Before Installation

- Inspect the product. If there is any damage or missing parts, do not install the product and contact your place of purchase.
- Ensure the surface you wish to mount the rail to is either solid or has adequate in-wall support.
- Ensure the surface you wish to mount the rail to is flat.
- Check that there will be adequate clearance for the rail to be folded up once installed.
- Nurse call cabling should be installed by a qualified electrician.

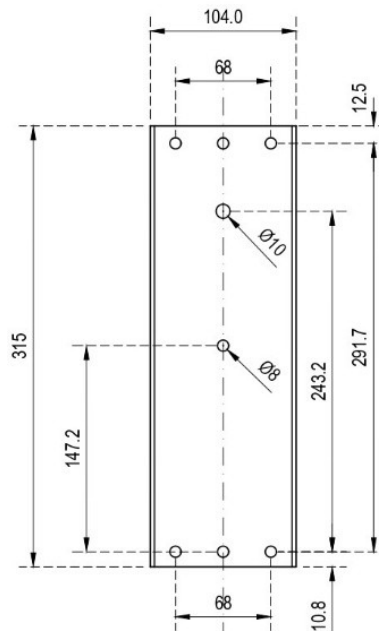

Installation Instructions

Placement may be subject to regulations/codes and architectural drawings. Correct positioning is the responsibility of the installer.

1. Fold the rail upwards to locate the mounting holes.
2. Lift the rail to the mounting surface based on the desired installation height and location of studs, and checking that all sides are level, mark the seven mounting points on the surface through the holes.

The eighth hole (closest to the hinge) is for the nurse call cabling to be fed through.

3. Secure the rail to the wall using appropriate fixings (not supplied).
4. Test rail for correct operation.
It should lift easily from a horizontal position to a vertical position without falling down.

Apply a small amount of pressure to pull the rail from the vertical position and lower it to the horizontal position, ready for use.


The tension of the hinge can be adjusted using a hex wrench to tighten or loosen the nut on the side of the rail.



Nurse Call Button Fit-Off

The nurse call holder is sized to suit most standard Australian electrical wall plates with 85mm centre-to-centre screw fixing points.

1. The holder can be swivelled from the left-hand to the right-hand side by loosening the grub screw underneath the holder and then re-tightening the grub screw once correctly positioned.
2. Attach the nurse call cabling to the wire protruding from the rail near the hinge.
3. Feed and pull the cabling through using the wire until it comes out of the holder.

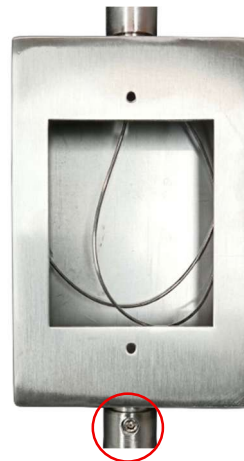

Technical recommendations and installation advice from Metlam Australia Pty Ltd are general in nature and provided in good faith without warranty. Metlam disclaims all liability for errors, omissions, or damage resulting from improper installation or maintenance. All products must be installed by qualified tradespeople and maintained by competent personnel. It is the installer's responsibility to conduct thorough pre-installation assessments; Metlam Australia will not enter into post-installation disputes regarding positioning, functionality, or external factors affecting performance.

The photographs and line drawings of the products presented above are representational only.
Metlam Australia Pty Ltd reserves the right to, and from time to time, make changes and improvements in design and dimensions.